

Instruments

Dosing table

Instructions to prepare a ready-to-use solution of an instrument cleaning or disinfectant product. The quantity of use solution consists of concentrate and tap water (e.g. to receive 10 litres of a 2 % solution, you have to add 200 millilitres of concentrate to 9.8 litres of tap water). Standing time according to manufacturer's recommendations and guidelines.

Concentrate in %	0.5 %	1 %	1.5 %	2 %	3 %	4 %	5 %
1 litre	5 ml	10 ml	15 ml	20 ml	30 ml	40 ml	50 ml
2 litres	10 ml	20 ml	30 ml	40 ml	60 ml	80 ml	100 ml
3 litres	15 ml	30 ml	45 ml	60 ml	90 ml	120 ml	150 ml
4 litres	20 ml	40 ml	60 ml	80 ml	120 ml	160 ml	200 ml
5 litres	25 ml	50 ml	75 ml	100 ml	150 ml	200 ml	250 ml
6 litres	30 ml	60 ml	90 ml	120 ml	180 ml	240 ml	300 ml
7 litres	35 ml	70 ml	105 ml	140 ml	210 ml	280 ml	350 ml
8 litres	40 ml	80 ml	120 ml	160 ml	240 ml	320 ml	400 ml
9 litres	45 ml	90 ml	135 ml	180 ml	270 ml	360 ml	450 ml
10 litres	50 ml	100 ml	150 ml	200 ml	300 ml	400 ml	500 ml
15 litres	75 ml	150 ml	225 ml	300 ml	450 ml	600 ml	750 ml
20 litres	100 ml	200 ml	300 ml	400 ml	600 ml	800 ml	1000 ml
30 litres	150 ml	300 ml	450 ml	600 ml	900 ml	1200 ml	1500 ml

